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
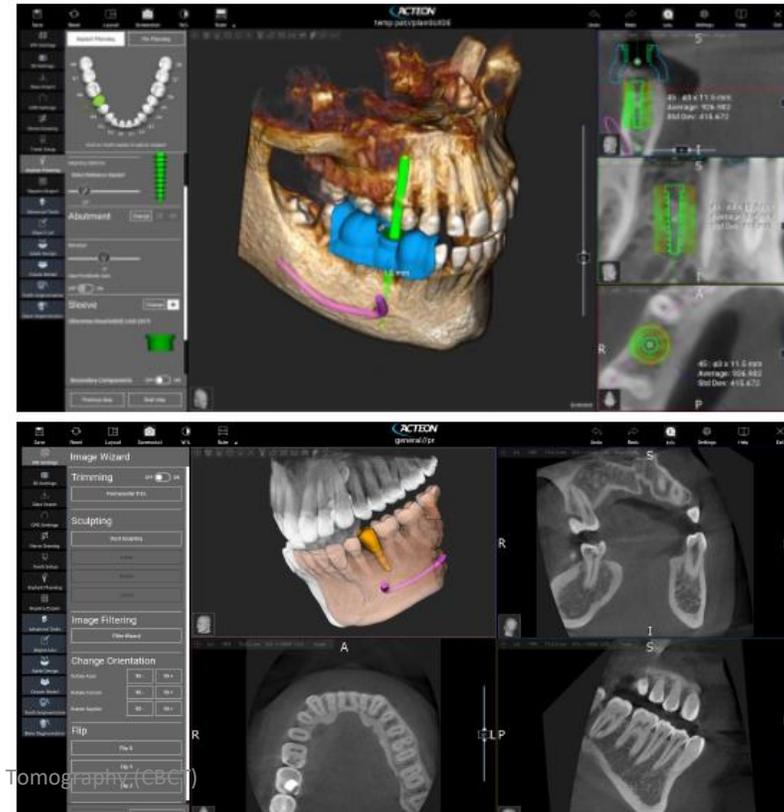
The new version on AIS 3D App comes with 3 types of licenses:



- START** – viewer mode
- PRO** – for implant planning and reports – lifetime license
- DESIGN** – guide design functionalities – annual license / **ANNUAL FEES to be paid yearly** .

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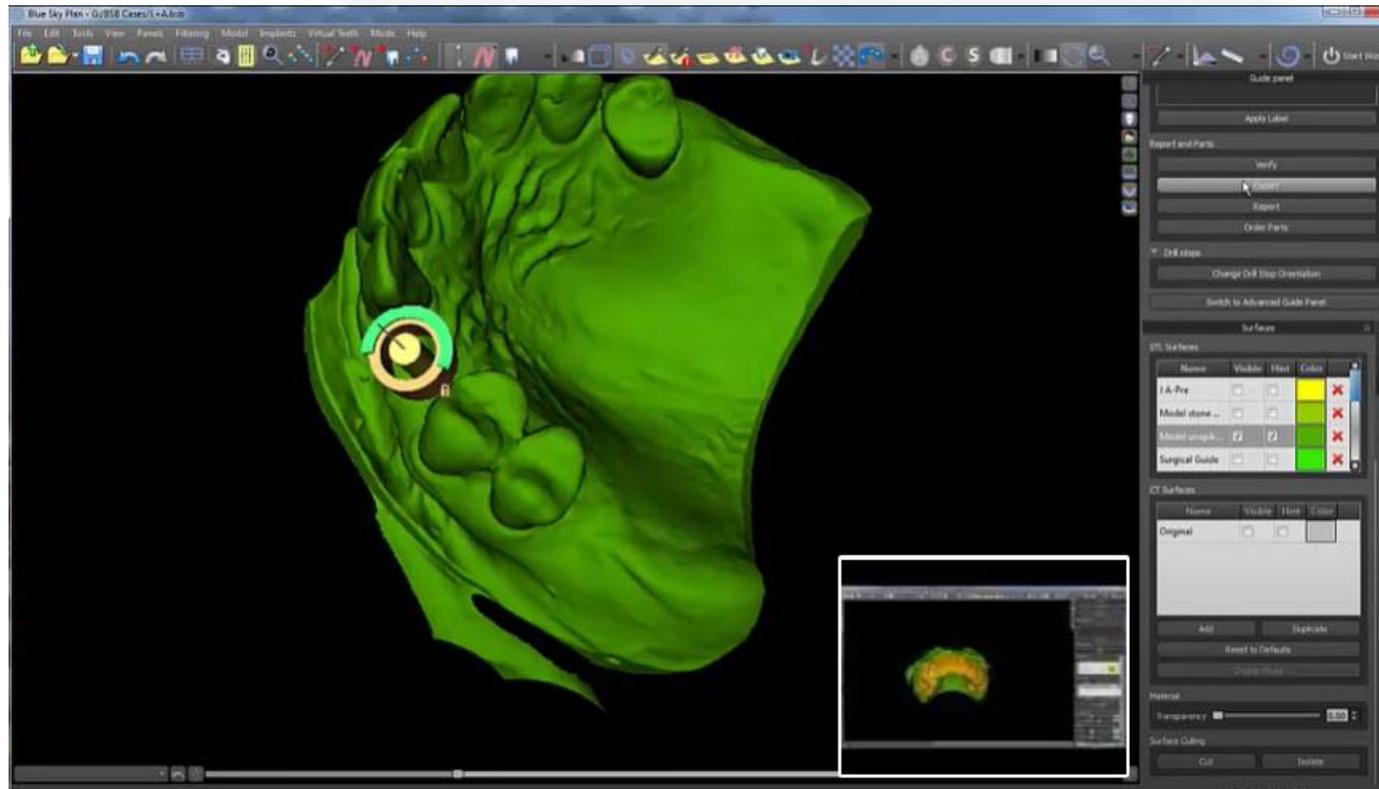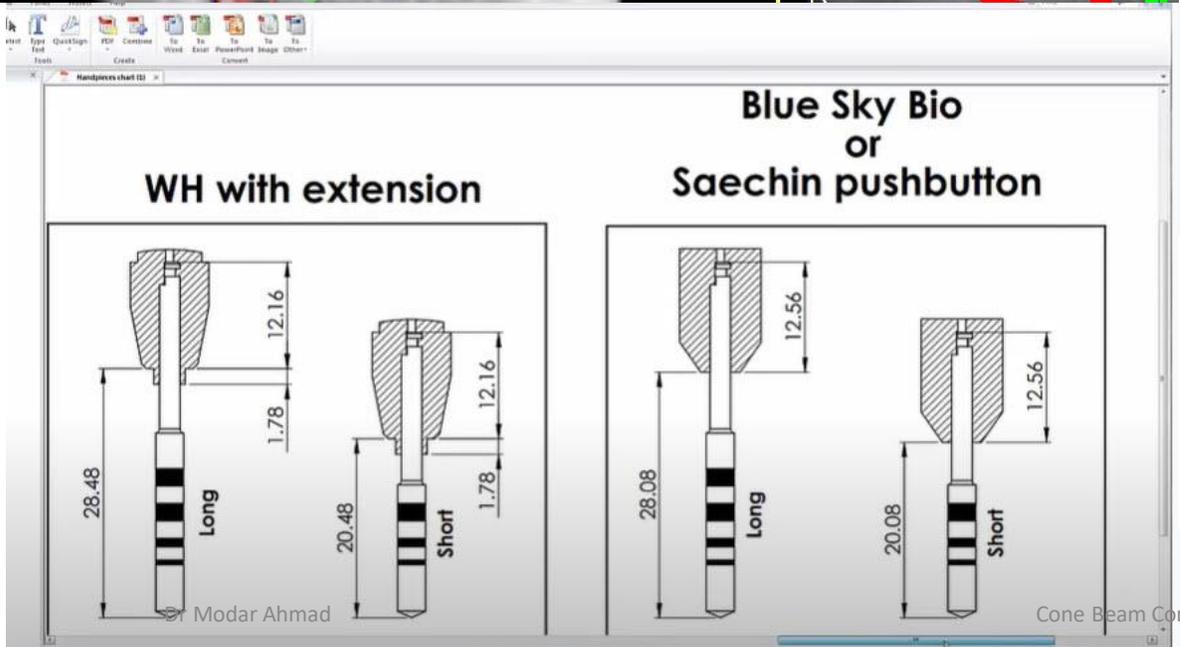
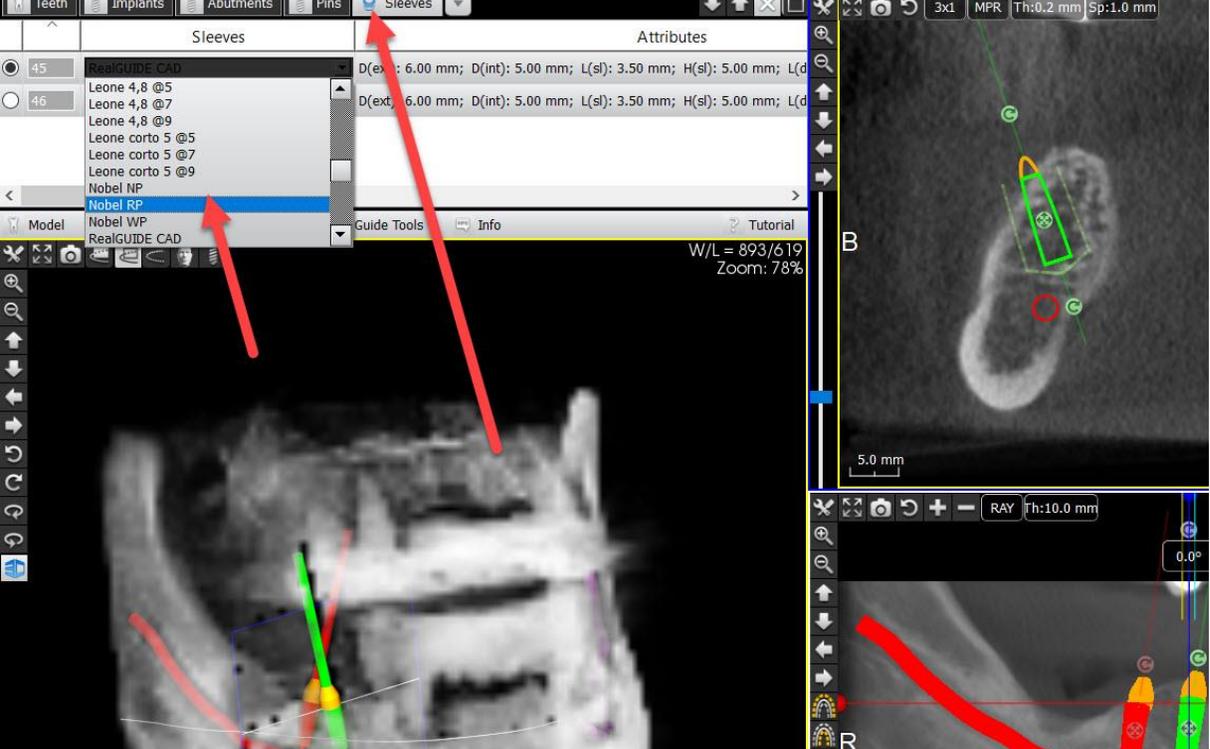
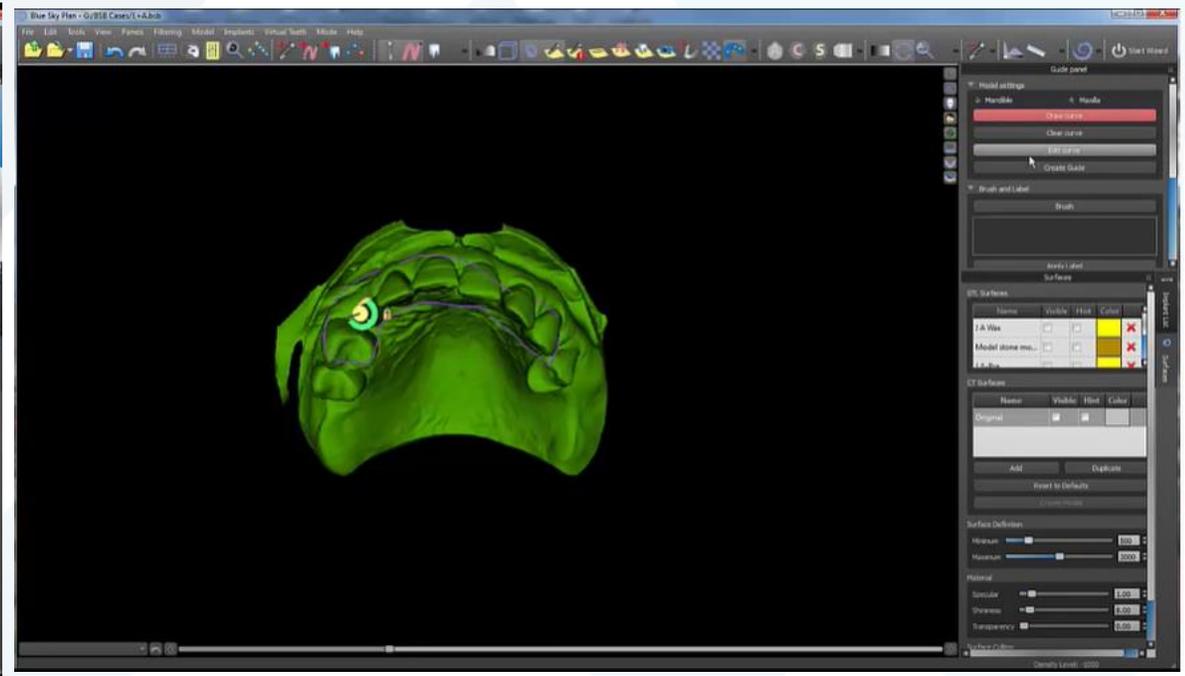
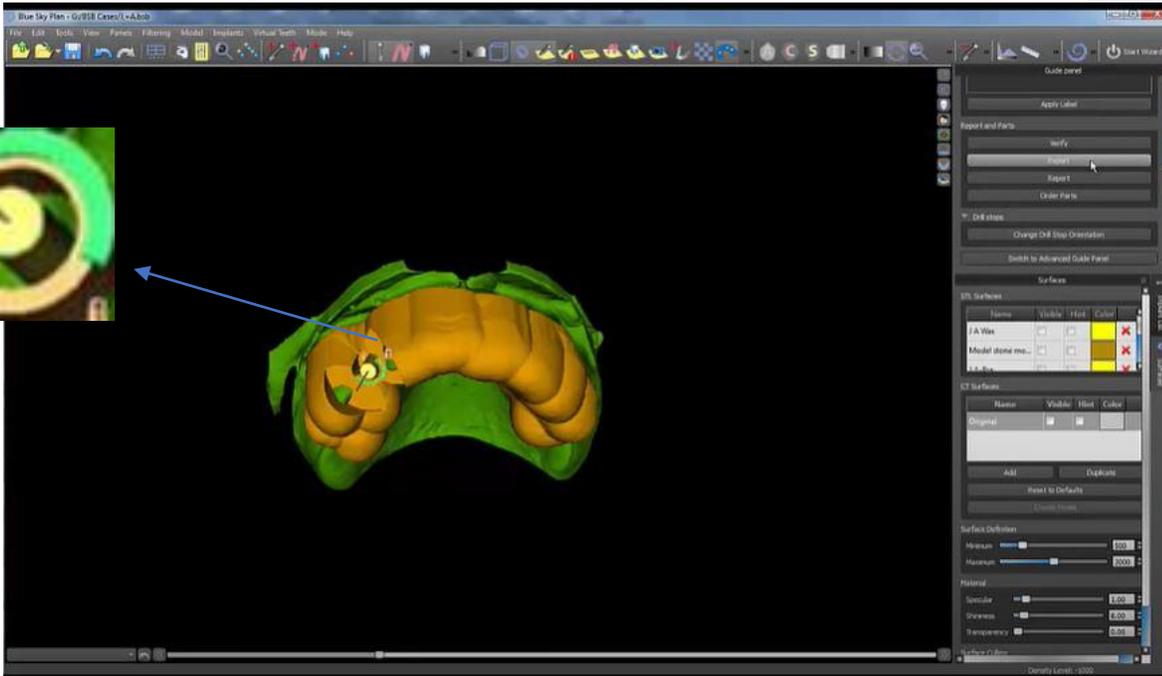
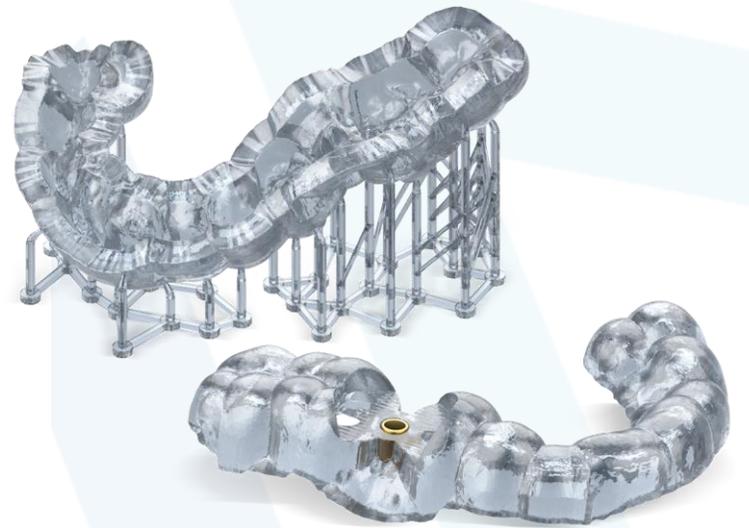
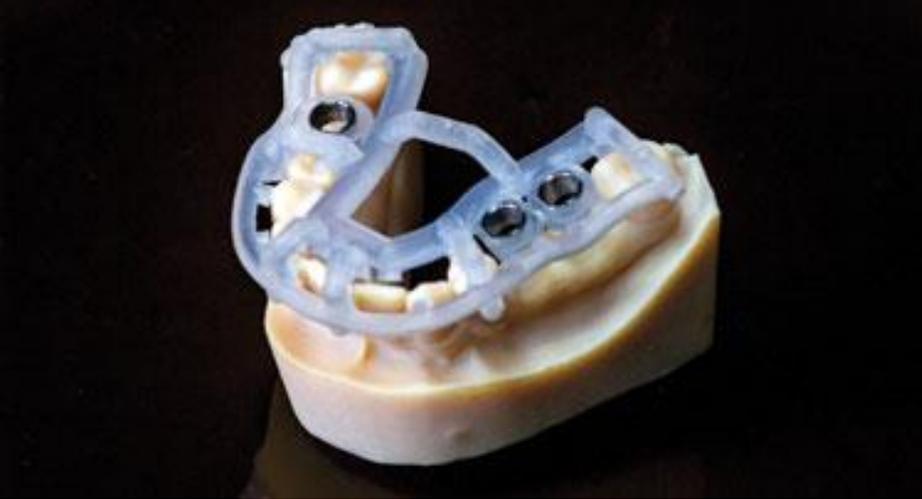
# How to use Bio3 Navigation Surgical Kit

## 3<sup>rd</sup> Stage Surgical template implant placement

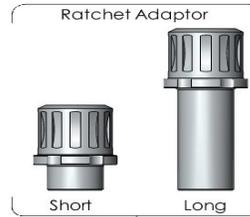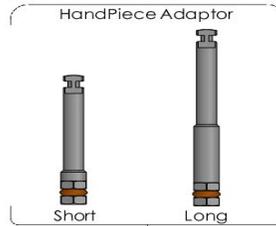**HandPiece Adaptor.**  
Is used to install the implant with a surgical tip. The HandPiece Adaptor connects to the surgical handpiece and fits into the internal Mounter hexagon. Screw the implant into the prepared implant site after.





**Ratchet Adaptor.**  
Is used for implant placement using Ratchet Wrench RR. The Ratchet Adapter connects to the Ratchet Wrench RR and connects to the Mounter outer hex. Screw the implant into the prepared implant site after.

**Mounter Platform Standard.**  
Is used to install implants with a diameter of 3.3 and 3.8 mm with the Standard platform. Mounter is inserted into the implant and fixed with the abutment fixation screw. The screw is installed using the key of a screwdriver SU-9 or SU-18.

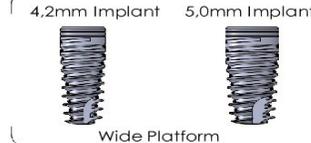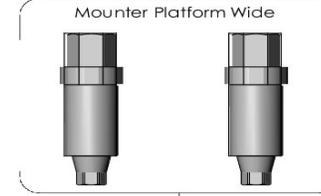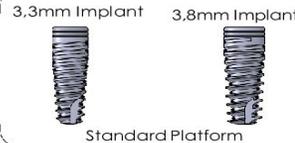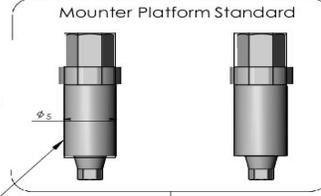Screw for Abutment SU-9

**Mounter Platform Wide.**  
used to install implants with a diameter of 4.2 and 5.0 mm with the Wide platform. Mounter is inserted into the implant and fixed with the abutment fixation screw. The screw is installed using the key of a screwdriver SU-9 or SU-18.

SU-9 Screw for Abutment

**Guide shank.** Each mounter has a 5 mm diameter guide part that matches the guide sleeve

**Implant placement.**  
After the bone bed has been prepared, the implant is installed using Mounter. For implants with a diameter of 3.3 and 3.8 mm, which have a Standard platform, Mounter Platform Standard is used. For 4.2 and 5.0 mm implants with the Wide - Mounter Platform Wide. The implant connected to the Mounter is inserted into the implant site through the guide sleeve.

## 2<sup>nd</sup> Stage

Preparation of the implant site according to the surgical template.

Drill 6 mm

**Start** with a short drill with a diameter of 2 mm and a length of 6 mm. The drill serves to mark the first hole, which is a marker for subsequent drillings.

Drill  $\varnothing$  2.0 mm

**Start Drill** is used after the first short 6 mm drill. It serves to set the required depth of the implant site under the appropriate length of the implant

Drill  $\varnothing$  2.8 mm

**Finishing drill** is used for implants with a diameter of 3 mm

Drill  $\varnothing$  3.2 mm

**Finishing drill** is used for implants with a diameter of 3.3 mm

Drill  $\varnothing$  3.65 mm

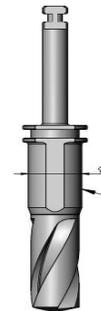
**Finishing drill** is used for implants with a diameter of 3.8 mm

Drill  $\varnothing$  4.0 mm

**Finishing drill** is used for implants with a diameter of 4.2 mm

Drill  $\varnothing$  4.7 mm

**Finishing drill** is used for implants with a diameter of 5.0 mm

**Guide shank.** Each drill has a diameter of 5 mm, which coincides with the guide sleeve

## 1<sup>st</sup> Stage

Design and creation of a surgical template.

You can design a surgical template using ReplikaCAD, Solid or BluskyPlan, which contains Bio3 digital libraries.

Cone Beam Computed Tomography

Drill Guide D5 mm (DG-D5)

**Guide sleeve** is installed in the manufactured template and serves to guide surgical drills and implant installation

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